

Rheumatoid Arthritis

ichroma™ RF IgM

INTENDED USE

ichroma™ RF IgM is a fluorescence Immunoassay (FIA) for the quantitative determination of RF IgM in human whole blood / serum / plasma. It is useful as an aid in management and monitoring of rheumatoid arthritis.

For *in vitro* diagnostic use only.

INTRODUCTION

Rheumatoid arthritis (RA) is the most common chronic autoimmune arthritis worldwide, leading to disability and substantial economic costs. It is a chronic and systemic inflammatory disorder that may affect many tissues and organs, but principally attacks synovial joints. About 1% of the world's population is afflicted by rheumatoid arthritis, women three times more often than men. Onset is most frequent between the ages of 40 and 50, but people of any age can be affected. It can be a disabling and painful condition, which can lead to substantial loss of functioning and mobility if not adequately treated.

PRINCIPLE

The test uses a sandwich immunodetection method; the detector antibody in buffer binds to antigen in sample, forming antigen-antibody complexes, and migrates onto nitrocellulose matrix to be captured by the other immobilized-antibody on test strip.

The more antigen in sample forms the more antigen-antibody complex and leads to stronger intensity of fluorescence signal on detector antibody, which is processed by instrument for ichroma™ tests to show RF IgM concentration in sample.

COMPONENTS

ichroma™ RF IgM consists of 'Cartridges', 'Detection Buffer Tubes', 'Sample collectors' and an 'ID chip'.

- The cartridge contains a test strip, the membrane which has Human Immunoglobulin at the test line, while KLH at the control line.
- Each cartridge is individually sealed in an aluminum foil pouch containing a desiccant. 25 sealed cartridges are packed in a box which also contains an ID chip and 25 sample collectors.
- The detection buffer contains anti human Immunoglobulin-fluorescence conjugate, anti KLH-fluorescence conjugate, bovine serum albumin (BSA) as a stabilizer and sodium azide in phosphate buffered saline (PBS) as a preservative.
- The detection buffer is dispensed in a tube. 25 detection buffer tubes are packaged in a box and further packed in a Styrofoam box with ice-pack for the shipment.

WARNINGS AND PRECAUTIONS

- For *in vitro* diagnostic use only.
- Carefully follow the instructions and procedures described in this 'Instructions for use'.
- Use only fresh samples and avoid direct sunlight.
- Lot numbers of all the test components (cartridge, ID chip and detection buffer) must match each other.
- Do not interchange the test components between different lots or use the test components after the expiration date, either of which might yield misleading of test result(s).
- Do not reuse. A detection buffer tube should be used for processing one sample only. So should a cartridge.

- The cartridge should remain sealed in its original pouch before use. Do not use the cartridge, if it is damaged or already opened.
- Frozen sample should be thawed only once. For shipping, samples must be packed in accordance with the regulations. Sample with severe hemolytic and hyperlipidemia cannot be used and should be recollected.
- Just before use, allow the cartridge, detection buffer and sample to be at room temperature for approximately 30 minutes.
- **ichroma™ RF IgM** as well as the instrument for ichroma™ tests should be used away from vibration and/or magnetic field. During normal usage, it can be noted that instrument for ichroma™ tests may produce minor vibration.
- Used detection buffer tubes, pipette tips and cartridges should be handled carefully and discarded by an appropriate method in accordance with relevant local regulations.
- An exposure to larger quantities of sodium azide may cause certain health issues like convulsions, low blood pressure and heart rate, loss of consciousness, lung injury and respiratory failure.
- **ichroma™ RF IgM** will provide accurate and reliable results subject to the following conditions.
 - Use **ichroma™ RF IgM** should be used only in conjunction with instrument for ichroma™ tests.
 - Any anticoagulants other than EDTA should be avoided.

STORAGE AND STABILITY

- The cartridge is stable for 20 months (while sealed in an aluminum foil pouch) if stored at 4 - 30°C.
- The detection buffer dispensed in a tube is stable for 20 months if stored at 2 - 8°C.
- After the cartridge pouch is opened, the test should be performed immediately.

LIMITATION OF THE TEST SYSTEM

- The test may yield false positive result(s) due to the cross-reactions and/or non-specific adhesion of certain sample components to the capture/detector antibodies.
- The test may yield false negative result. The non-responsiveness of the antigen to the antibodies is most common where the epitope is masked by some unknown components, so as not to be detected or captured by the antibodies. The instability or degradation of the antigen with time and/or temperature may cause the false negative as it makes antigen unrecognizable by the antibodies.
- Other factors may interfere with the test and cause erroneous results, such as technical/procedural errors, degradation of the test components/reagents or presence of interfering substances in the test samples.
- Any clinical diagnosis based on the test result must be supported by a comprehensive judgment of the concerned physician including clinical symptoms and other relevant test results.

MATERIALS SUPPLIED

REF CFPC-39

Components of **ichroma™ RF IgM**

- Cartridge Box:
 - Cartridges 25
 - Sample collectors 25
 - ID Chip 1
 - Instruction For Use 1
- Box containing Detection Buffer tubes
 - Detection Buffer tubes 25

MATERIALS REQUIRED BUT SUPPLIED ON DEMAND

Following items can be purchased separately from **ichroma™ RF IgM**. Please contact our sales division for more information.

- Instrument for **ichroma™** tests
 - **ichroma™ Reader** [REF](#) FR203
 - **ichroma™ D** [REF](#) 13303
- **ichroma™ Printer** [REF](#) FPRR007
- **ichroma™ RF IgM Control** [REF](#) CFPO-17

SAMPLE COLLECTION AND PROCESSING

The sample type for **ichroma™ RF IgM** is human whole blood / serum / plasma.

- It is recommended to test the sample within 24 hours after collection.
- The serum or plasma should be separated from the clot by centrifugation within 3 hours after the collection of whole blood. If longer storage is required, e.g. if the test could not be performed within 24 hours, serum or plasma should be immediately frozen below -20°C. The freezing storage of sample up to 3 months does not affect the quality of results.
- However, the whole blood sample should not be kept in a freezer in any case.
- Once the sample was frozen, it should be used one time only for test, because repeated freezing and thawing can result in the change of test values.
- Fingertip blood sample should be collected as follows:
 - Position the hand such that the palm should be facing upwards.
 - Blood should be normally drawn from the middle or ring finger of the non-dominant hand. Apply intermittent pressure on the least calloused finger towards its tip.
 - Wipe the fingertip clean with an alcohol pad.
 - Allow the finger to dry completely because blood will not form a drop if the puncture site is moist. The residual alcohol at the fingertip may also dilute the blood sample thereby affecting the test result.
 - Hold the finger and puncture the fingertip by firmly pressing a new sterile lancet against it at an off-center position.
 - Wipe away the first drop of blood with a sterile gauze pad or cotton ball.
 - Massage the finger towards its tip to form a new drop of blood. Blood will flow easily if the finger is held lower than the elbow.
 - Hold the handle of a capillary tube and touch the mouth of the capillary to the drop of blood.
 - Let the blood fill the capillary tube completely.
 - It may be sometimes necessary to massage the finger again for an additional drop of blood for filling the capillary tube.

TEST SETUP

- Check the contents of **ichroma™ RF IgM**: Sealed Cartridge, Detection Buffer Tubes, Sample collectors and ID Chip.
- Ensure that the lot number of the cartridge matches that of the ID chip as well as the detection buffer.
- Keep the sealed cartridge (if stored in refrigerator) and the detection buffer tube at room temperature for at least 30 minutes just prior to the test. Place the cartridge on a clean, dust-free and flat surface.
- Turn on the instrument for **ichroma™** tests.
- Insert the ID Chip into the ID chip port of the instrument for **ichroma™** tests.
- Press the 'Select' button on the instrument for **ichroma™** tests. (Please refer to the 'Instrument for **ichroma™** tests Operation Manual' for complete information and operating instructions.)

TEST PROCEDURE

- 1) Make a puncture on the top of the detection buffer tube by inserting an empty sample collector.
- 2) Draw 10 µL (Human whole blood) of sample using a sample collector. If the sample is not whole blood, transfer 5 µL (Human serum / plasma / control) with a pipette to detection buffer tube. (If necessary, wipe out the excess blood outside of the capillary on the sample collector with paper towel.)
- 3) Assemble the sample collector and the tube into one.
- 4) Shake the 10 times or more until the sample out of the sample collector by inversion. The mixture of buffer and the sample has to be used within 30 seconds.
- 5) Remove the cap off the top of assembled tube. Discard two drops of sample mixture onto the paper towel before loading.
- 6) Load only two drops of the mixture onto the sample well of the cartridge.
- 7) Leave the sample-loaded test cartridge at room temperature for 5 minutes.
- 8) To scan the sample-loaded cartridge, insert it into the cartridge holder of the instrument for **ichroma™** tests. Ensure proper orientation of the cartridge before pushing it all the way inside the cartridge holder. An arrow has been marked on the cartridge especially for this purpose.
- 9) Press 'Select' button on the instrument for **ichroma™** tests to start the scanning process.
- 10) Instrument for **ichroma™** tests will start scanning the sample-loaded cartridge immediately.
- 11) Read the test result on the display screen of the instrument for **ichroma™** tests.

INTERPRETATION OF TEST RESULT

- Instrument for **ichroma™** tests calculates the test result automatically and displays RF IgM concentration of the test sample in terms of IU/mL.
- The cut-off (reference value)
 - 20 IU/mL
- Working range : 10 – 200 IU/mL

QUALITY CONTROL

- Quality control tests are a part of the good testing practice to confirm the expected results and validity of the assay and should be performed at regular intervals.
- The control tests should be performed immediately after opening a new test lot to ensure the test performance is not altered.
- Quality control tests should also be performed whenever there is any question concerning the validity of the test results.
- Control materials are not provided with **ichroma™ RF IgM**. For more information regarding obtaining the control materials, contact **Boditech Med Inc.'s Sales Division for assistance**. (Please refer to the instruction for use of control material.)


PERFORMANCE CHARACTERISTICS

- **Specificity**: There, in test samples, are biomolecules such as Bilirubin, lipid, hemoglobin, AFP, CEA, and albumin in higher concentration than their normal physiological levels. But this doesn't interfere with the **ichroma™ RF IgM** test measurements, nor occurs any significant cross-reactivity.
- **Precision**: The intra-assay precision was calculated by one evaluator, who tested different concentration of control standard ten times each with three different lots of **ichroma™ RF IgM**. The inter-assay precision was confirmed by 3 different evaluators with 3 different lots, testing 10 times each different concentration.

RF IgM [IU/mL]	Intra-assay		Inter-assay	
	Mean	CV (%)	Mean	CV (%)
10	10.3	3.7	10.3	3.1
20	20.7	2.7	20.5	3.1

40	41.1	2.5	40.7	3.7
100	100.2	3.3	101.0	4.0
150	149.2	2.0	152.7	3.6
200	194.0	1.8	192.0	2.2



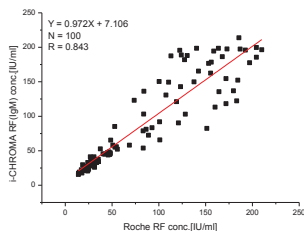








- Comparability: RF IgM** concentrations of 100 clinical samples were quantified independently with **ichroma™ RF IgM** and Roche modular as per prescribed test procedures. Test results were compared and their comparability was investigated with linear regression and coefficient of correlation (R). Linear regression and coefficient of correlation between the two tests were $Y=0.972X + 7.106$ and $R = 0.843$ respectively.



REFERENCES

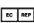- Franziska Renge. Immediate determination of ACPA and rheumatoid factor - a novel point of care test for detection of anti-MCV antibodies and rheumatoid factor using a lateral-flow immunoassay. 2010. Arthritis Res Ther. 12:R120.
- Hiroshi Oka, Shunsei Hirohata. Quantitative determination of IgM-rheumatoid factor by enzyme immunoassay - Standardization using a serum from a rheumatoid arthritis patient. 1990. 188:147-159.
- Williams RC Jr. Autoimmune mechanisms involved in the pathogenesis of rheumatoid arthritis. 1996. Adv Dent Res. 10(1):47-51
- Gioud-Paquet M, Auvinet M, Raffin T, Girard P. IgM rheumatoid factor (RF), IgA RF, IgE RF, and IgG RF detected by ELISA in rheumatoid arthritis. 1987. Ann Rheum Dis. 46(1):65-71.
- Kawashiri SY, Kawakami A, Ueki Y, Imazato T, Iwamoto N, Fujikawa K, Aramaki T, Tamai M, Nakamura H, Origuchi T, Ida H, Eguchi K. Decrement of serum cartilage oligomeric matrix protein (COMP) in rheumatoid arthritis (RA) patients achieving remission after 6 months of etanercept treatment: comparison with CRP, IgM-RF, MMP-3 and anti-CCP Ab. 2010. Joint Bone Spine. 77(5):418-20.

Note: Please refer to the table below to identify various symbols

	Read instructions for use
	Use by
	Batch code
	Catalog number
	Caution
	Manufacturer
	Authorized representative of the European Community
	In vitro diagnostic medical device
	Temperature limit
	Do not reuse
	This product fulfills the requirements of the Directive 98/79/EC on in vitro diagnostic medical devices

For technical assistance; please contact:

Boditech Med Inc.'s Technical Services

Tel: +82 33 243-1400

E-mail: sales@boditech.co.kr



Boditech Med Incorporated

43, Geodudanji 1-gil, Dongnae-myeon,
Chuncheon-si, Gang-won-do 200-883
Republic of Korea

Tel: +(82) -33-243-1400

Fax: +(82) -33-243-9373

www.boditech.co.kr



Obelis s.a

Bd. Général Wahis 53,
1030 Brussels, BELGIUM

Tel: +(32) -2-732-59-54

Fax: +(32) -2-732-60-03

E-Mail: mail@obelis.net

